



## StarMed Ventumask Basic

For CPAP therapy with Venturi flow driver and adjustable PEEP valve



Critical Care ▪ Patient Interfaces



## StarMed Ventumask Basic with Venturi flow driver and adjustable PEEP valve

The Ventumask Basic is a unique design for CPAP therapy that is simply activated by connecting to an appropriate oxygen supply. It can be used in a hospital environment or pre-hospital use, for example in an ambulance.

### Advantages and Technical Features

- Ventumask Basic is a complete and easy to use system, where the oxygen supply tube, a Venturi flow driver and a PEEP valve are integral to the CPAP mask
- The integrated Venturi flow driver is operated by a single oxygen flowmeter to generate high patient flows while consuming a reduced amount of compressed gas
- The built-in adjustable PEEP valve is designed exclusively for Ventumask Basic and the administration to the patient is facilitated at some pre-settings: 5 - 7.5 - 10 - 12.5 cmH<sub>2</sub>O
- O<sub>2</sub>% of 40% can be delivered at each PEEP level
- The user is guided by a performance table integral to the product, to be aware of patient flow at each PEEP settings



#### The kit is composed of:

CPAP mask with safety valve and port to monitor actual O<sub>2</sub>% or pressure

Venturi flow driver integral to the mask

Adjustable headgear included

Adjustable PEEP valve designed for Ventumask Basic facilitates some pre-calibrated pressure settings

Integral oxygen tube

### Ventumask Basic performance: for CPAP therapy with Venturi flow driver and adjustable PEEP valve

Requirement		Setting	Flow delivered
O <sub>2</sub> % approx	PEEP cmH <sub>2</sub> O	Flow O <sub>2</sub> L/min	Flow approx L/min
40%	5	12	57
	7.5	12	52
	10	12	47
	12.5	12	42

Code	Description	Size	Box Qty.
CV203S	Ventumask Basic CPAP mask with integral Venturi flow driver and adjustable PEEP valve	Small	5
CV203M/L	Ventumask Basic CPAP mask with integral Venturi flow driver and adjustable PEEP valve	Medium / Large	5
CV203XL	Ventumask Basic CPAP mask with integral Venturi flow driver and adjustable PEEP valve	Extra Large	5



[www.intersurgical.com.au/info/starmed](http://www.intersurgical.com.au/info/starmed)

Make an enquiry

IS1.45\_AU • Issue 2 06.16



Unit 4, 151 Beauchamp Road, Matraville, NSW, 2036

T: +61 (0) 2 8048 3300 [info@intersurgical.com.au](mailto:info@intersurgical.com.au) [www.intersurgical.com.au](http://www.intersurgical.com.au)

UK • Ireland • France • Germany • Spain • Portugal • Italy  
 Netherlands • Sweden • Denmark • Lithuania • Russia  
 Czech Republic • Turkey • South Africa • China • Japan  
 Taiwan • Philippines • USA • Colombia • Australia



**Please think before you print**  
 Save energy and paper.  
 If you must print this e-mail  
 please print it double sided

Intersurgical Ltd is certified to ISO 14001, ISO 9001 and ISO 13485