




Control-Cric™

Surgical cricothyrotomy kit

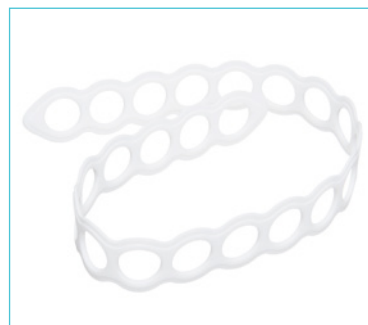

Airway Management ▪ Pulmodyne®



Control-Cric™ surgical cricothyrotomy kit

The Control-Cric is packaged in 3 clear steps, guiding the provider through an otherwise chaotic procedure.

Step 1 is the Cric-Knife™, a 10 mm dual-sided blade with an integrated sliding tracheal hook. This allows the provider to maintain control of the airway instead of reaching for additional tools and potentially losing the airway.

Step 2 is the Cric-Key™, a 5.5 mm cuffed tube with an integrated introducer. The introducer gives tactile feedback of the tracheal rings, confirming placement.

Step 3 is the neck strap, a flexible silicone strap to slide easily over the tube to maintain position during ventilation.



Compact 3 step packaging
Rapid deployment to oxygenate/ventilate
BVM wedge included




Cric-Knife

- 10 mm dual-sided blade
- Integrated sliding tracheal hook
- Designed to grip like a pencil



Cric-Key

- 5.5 mm cuffed tube
- Integrated introducer
- Provides tactile feedback of the tracheal rings to avoid false passage

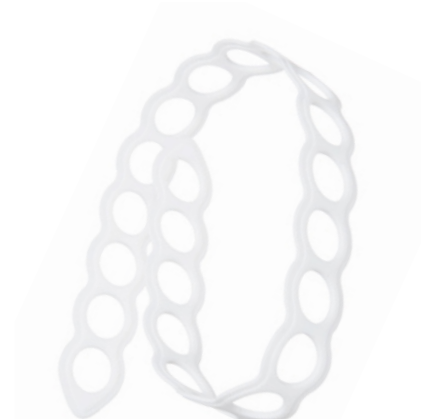

Neck strap

- Secure the tube after procedure
- Fits on 22 mm connection

Code	Description	Box Qty.
313-4567	Control-Cric kit sterile	5 
313-4568	Control-Cric training kit	10



Sterile

Control-Cric, Cric-Key and Cric-Knife are manufactured by Pulmonary Inc and distributed by Intersurgical Ltd.

Please think before you print Save energy and paper. If you must print this information sheet please print it double sided.

IS6.21 ▪ Issue 1 10.25



Intersurgical Ltd, Crane House, Molly Millars Lane, Wokingham, Berkshire, RG41 2RZ, UK
T: +44 (0)118 965 6300 info@intersurgical.com www.intersurgical.com

UK ▪ Australia ▪ Benelux ▪ Brazil ▪ Canada ▪ China ▪ Colombia ▪ Czech Republic ▪ Denmark ▪ France ▪ Germany ▪ India ▪ Ireland ▪ Italy
Japan ▪ Lithuania ▪ Malaysia ▪ Mexico ▪ Norway ▪ Philippines ▪ Portugal ▪ Russia ▪ South Africa ▪ Spain ▪ Sweden ▪ Taiwan ▪ Turkey ▪ USA