



Cricalot

High repetition task training kit for surgical cricothyrotomy



Airway Management ▪ Surgical cricothyrotomy



Cricalot

Cricalot is a high repetition task trainer for surgical cricothyrotomy. Cricalot addresses the need for an anatomically accurate, realistic, low-cost training kit for cricothyrotomy allowing clinicians to routinely practice this rarely occurring, potentially lifesaving procedure.



Anatomically accurate

Each Cricalot kit contains a pair of models, one female and one male, with anatomically accurate cricoid cartilage giving familiarity with both female and male anatomy. When opened, the outer case of each model acts as a sternum, allowing stabilisation of the larynx and operator's cutting hand, both critical elements of the cricothyrotomy procedure.

Skin analogue

200 skins and 2 rolls of cotton padding are provided with each Cricalot. Cotton padding can be placed into a skin pouch to achieve the desired thickness. Artificial blood may also be added to the skin pouch to create a more realistic model.

Easy to use and cost-effective

Skins are cost effective and feature an adhesive back, allowing easy application to each model. After use, they are quick to remove and replace, with each model rapidly available to be used again.

Cricalot pack contents:

- 1 female model
- 1 male model
- 200 skins
- 2 rolls of cotton padding
- 6 spare tracheas



Feature and benefits

Criclot has a range of features giving significant benefits to users.

Sternum

The sliding outer case of each Criclot model acts as a sternum when open, allowing stabilisation of the larynx and operators cutting hand, both critical elements of the cricothyrotomy procedure

Skins

Easily changeable, cost-effective, plastic pouches allowing the addition of artificial blood for a more realistic model

Female and male model

Each Criclot set contains two models, one female and one male, allowing training with an accurate representation of the cricoid cartilage in both females and males

Larynx

Can move side to side, allowing the 'laryngeal handshake' to identify the cricothyroid membrane

Trachea

The trachea has a softer feel, making it distinguishable from the cricoid cartilage. The trachea also has both external and internal rings

Rigid, durable outer case

Space to store all contents and any other equipment required

Compact, light design

Allows easy transportation and storage



Code	Description	Box Qty.
3131000	Criclot surgical cricothyrotomy trainer	1



Cricalot is ideal for use with the Control-Cric™ training kit (Code: 313-4568)

The Control-Cric training kit is a non-sterile, training version of the Control-Cric which allows familiarisation with the Control-Cric and its features as part of a regular training program. The Control-Cric training kit is easily distinguished from the Control-Cric with its distinctive outer packaging and yellow Cric-Key™ and Cric-Knife™.



Note: The Control-Cric training kit is not for human use.

Code	Description	Box Qty.
313-4568	Control-Cric training kit	10

Cricalot is manufactured by cricalot.com and distributed by Intersurgical Ltd. Control-Cric™, Cric-Key™ and Cric-Knife™ are manufactured by Pulmodyne Inc and distributed by Intersurgical Ltd.

Note: Not all products are available in all markets, please contact your local office for details.

IS6.29 • Issue 1 04.26



Intersurgical Ltd, Crane House, Molly Millars Lane, Wokingham, Berkshire, RG41 2RZ, UK
T: +44 (0)118 965 6300 info@intersurgical.com www.intersurgical.com

UK • Australia • Benelux • Brazil • Canada • China • Colombia • Czech Republic • Denmark • France • Germany • India • Ireland • Italy
Japan • Lithuania • Malaysia • Mexico • Norway • Philippines • Portugal • Russia • South Africa • Spain • Sweden • Taiwan • Turkey • USA