

MRI compatible Solus™

A range of laryngeal mask airways for use in
Magnetic Resonance Imaging



The MRI compatible Solus™



When MRI scanning it is important that no magnetic material, such as ferrous metal is incorporated in any of the equipment used. Even the smallest quantity of ferrous material can interfere with the fine calibration of the magnet and bring about distorted imagery.

The standard Solus range incorporates a high quality metal spring in its valve unit. To overcome problems associated with ferrous metal entering the MRI machine during the scanning process we have specifically designed an MRI Compatible Solus that is fitted with a non-ferrous valve.

The MRI compatible non-ferrous valve has been tested under laboratory conditions to ensure that when the valve plunger is removed it will still function perfectly.


The MRI compatible Solus has a yellow tinted pilot balloon and valve plunger, giving a clear indication of its non-ferrous properties. The packaging for the MRI compatible Solus range has an MRI logo to differentiate from the standard Solus products.



The MRI compatible Solus demonstrates Intersurgical's continued commitment to producing safe and reliable single use alternatives. The MRI compatible Solus is available in a range of 5 sizes and patient weights.

MRI compatible laryngeal mask airways

<div>8005</div> <div> </div>			
8005002	MRI compatible laryngeal mask Patient size: >70kg	size 5 Maximum inflation value: <40ml	20
8004002	MRI compatible laryngeal mask Patient size: 50-70kg	size 4 Maximum inflation value: <30ml	20
8003002	MRI compatible laryngeal mask Patient size: 30-50kg	size 3 Maximum inflation value: <20ml	20
8025002	MRI compatible laryngeal mask Patient size: 20-30kg	size 2½ Maximum inflation value: <14ml	20
8002002	MRI compatible laryngeal mask Patient size: 10-20kg	size 2 Maximum inflation value: <10ml	20

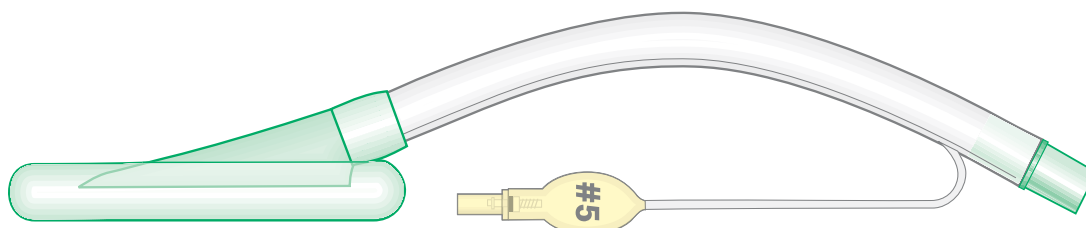

IS6.2 KW Issue 1 07.06



INTERSURGICAL®
COMPLETE RESPIRATORY SYSTEMS

United Kingdom (Head Office): Crane House, Molly Millars Lane, Wokingham, Berkshire, RG41 2RZ
T: +44 (0)118 9656 300 F: +44 (0)118 965 6356 info@intersurgical.com | www.intersurgical.com

France
info@intersurgical.fr
Portugal
info@intersurgical.pt

Deutschland
info@intersurgical.de
Nederland
info@intersurgical.nl

España
info@intersurgical-es.com
Lietuva
info@intersurgical.lt

Россия
info@intersurgical.ru
Česká Republika
info@intersurgical.cz

South Africa
info@intersurgical.co.za
Philippines
info@intersurgical.ph



Intersurgical manufacturing sites re FDA registered