

STARMed Ventumask

For CPAP therapy with Venturi flow driver and adjustable PEEP valve



StarMed Ventumask – for CPAP therapy with Venturi flow driver and adjustable PEEP valve

StarMed Ventumask is a unique design for CPAP therapy that is simply activated by connecting to an appropriate oxygen supply, offering:

- Convenience
- Reliability
- Ease of use
- Immediate control of the basic therapy parameters



Dual Flow Meter
Code No.: STS.15/30BS
Dual Flow meter 15/30 L/min with O₂ BS coupling



CE certified product

Table of Ventumask performance

| Requirement | | Setting | Flow Delivered |
|-----------------------------|---|-------------------------------------|-------------------|
| FiO ₂ (*) approx | Peep cm H ₂ O | Flow O ₂ l/min (A) + (B) | Flow approx l/min |
| 40% | 5 | 12 + 0 | 57 |
| | 7.5 | 12 + 0 | 52 |
| | 10 | 12 + 0 | 47 |
| | 12.5 | 12 + 0 | 42 |
| 50% | 5 | 10 + 12 | 58 |
| | 7.5 | 10 + 12 | 54 |
| | 10 | 10 + 10 | 47 |
| | 12.5 | 10 + 10 | 42 |
| Requirement | | Setting | Flow Delivered |
| FiO ₂ (*) approx | Peep cm H ₂ O | Flow O ₂ l/min (A) + (B) | Flow approx l/min |
| 60% | 5 | 8 + 20 | 60 |
| | 7.5 | 8 + 20 | 55 |
| | 10 | 8 + 20 | 50 |
| | 12.5 | 9 + 16 | 42 |
| 100% | Feed with a minimum O ₂ flow of 42 l/min | | |

Note – The use of flow meters with an output of less than 30 litres/minute on line B, will reduce the performance of the device above 50% FiO₂

| | | | | |
|--------------------|--|-----------------------|---|--|
| CV200M | CPAP mask with integral Venturi flow driver and adjustable PEEP valve | Small / Medium | 5 | |
| CV200L | CPAP mask with integral Venturi flow driver and adjustable PEEP valve | Large | 5 | |
| CV200XL | CPAP mask with integral Venturi flow driver and adjustable PEEP valve | Extra Large | 5 | |
| STS.15/30BS | Dual Flow Meter STS:15/30BS flow meter 15/30 L/min with O ₂ BS coupling | | 1 | |

Single packed. Minimum purchasing package: 5 units. Custom-packs are supplied on specific request of the customer.

IS1.23_UK • Issue 2 12.12



Crane House, Molly Millars Lane,
Wokingham, Berkshire, RG41 2RZ
T. +44 (0)118 9656 300
F. +44 (0)118 9656 356

UK
info@intersurgical.co.uk
Italy
info@intersurgical.it

España
info@intersurgical-es.com
Portugal
info@intersurgical.pt

Lietuva
info@intersurgical.lt
Россия
info@intersurgical.ru

France
info@intersurgical.fr
South Africa
info@intersurgical.co.za

Japan
info@intersurgical.co.jp
Nederland
info@intersurgical.nl



Manufactured by: Intersurgical S.p.A.